Distributed by:

ABLE2 EUROPE BV
Koperslagerij 15
4762 AR Zevenbergen
The Netherlands
www.able2.nl

VALENTINE INTERNATIONAL Ltd.
8th Floor, No. 149, Sec 2, Ta Tung Road
221 Hsichih District, New Taipei City
Taiwan R.O.C.

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FOR YOUR SAFETY AND COMFORT

Deluxe inflatable shampoo basin

Please inspect all parts carefully for cracks/breaks prior to use.

This inflatable shampoo basin supports the head and neck comfortably during the shampoo process. This product is intended to be used with the aid of a caregiver while the user is in bed lying flat.

Instructions for use:

- Inflate the shampoo basin with air by mouth or with an air pump (not included).
- Place the basin under user's head with the opening at the neck.
- Place the end of the drain hose into a waterproof waste receptacle. NOTE: Make sure the water receptacle is lower than the basin so water will drain properly. There is a drain plug provided in the bottom of the basin if needed.
- Wet hair, apply shampoo, and then rinse with water.
- When all water has drained out of the basin, remove the basin from under the user's head.
- Rinse and thoroughly dry basin after use. Keep inflated for several hours after use to assure it is completely dry before folding and storage.

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